





## "PROPER USE AND CARE GUIDE"

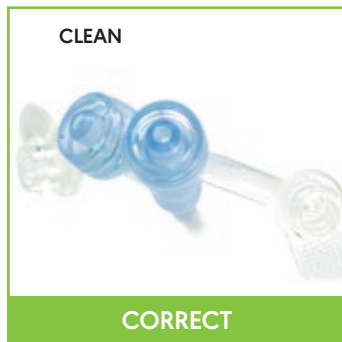

**1 CLEAN:** Avoid nutrition build up in connector threads and tethered caps. Clean regularly with warm soapy water per the IFU.

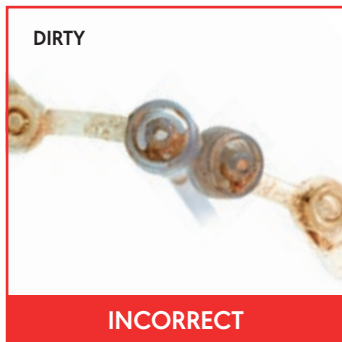

**2 CONNECT:** Make sure connection is secure to avoid leakage. Avoid over tightening the feed set connector. **STOP** when you feel a "snug" connection.

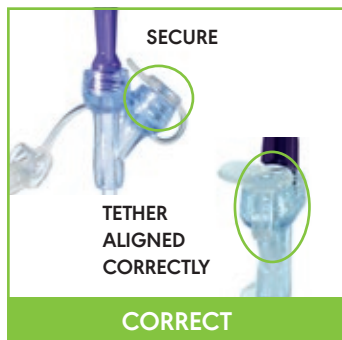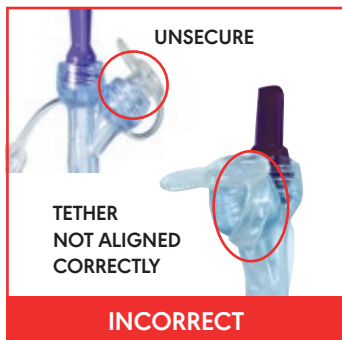



**3 CAP:** Ensure the tethered cap is securely closed by pushing or twisting it in. To close by twisting, push the cap firmly inside the port. Once locked, turn the cap within the port to ensure the strap is straight. A diagonal strap pulls on the cap creating tension, which may cause leakage.



**4 ALIGN:** Avoid misalignment and cross threading connections, especially with syringe. This is key to prevent cracking of the access port and potential damage of the extension set.



**5 OTHER CONNECTIONS:** Try to avoid hitting connections attached to the other port, especially when using an offset syringe.

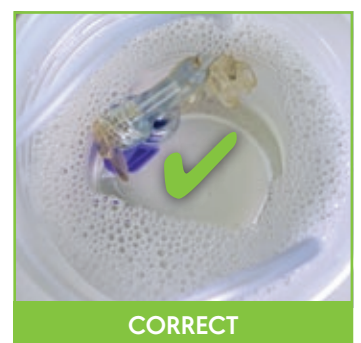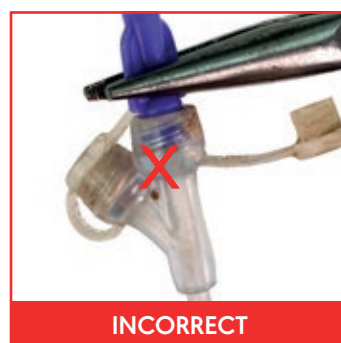

**6 DISCONNECT:** Avoid forcing connectors apart or disconnecting with pliers, clamps or other tools. If you are unable to disconnect due to build up, soak in warm soapy water.

## DAILY CARE

- Check the position of the MIC-KEY<sup>®</sup> G Feeding Tube, using pH-indicator paper<sup>1</sup>. This should be done each time you connect a new packaging of tube feeds, when there is any doubt whether the tube is in the correct position and at least 3 times a day. In case of doubt: confirm correct placement by X-ray. Always flush the tube with water before and after feeding and medication.
- Wash the MIC-KEY<sup>®</sup> extension set after every use with warm water and soap. Rinse thoroughly and allow to air dry. Store in air tight container or zip lock bag until next use. MIC-KEY<sup>®</sup> Extension Sets are disposables and should be replaced at least every 2 weeks.
- Change the MIC-KEY<sup>®</sup> G-tube every 3 months for optimal results<sup>2</sup>.

## MIC-KEY<sup>®</sup> FEEDING TUBE - THE ORIGINAL !

SIZE (OUTER DIAMETER)	STOMA LENGTH (Y)	PRODUCT CODE
12 Fr	From 0.8 to 4.0	8140-x-y
14 Fr	From 0.8 to 5.0	8140-x-y
16 Fr	From 0.8 to 5.0	8140-x-y
18 Fr	From 0.8 to 5.0	8140-x-y
20 Fr	From 0.8 to 5.0	8140-x-y
24 Fr	From 1.5 to 5.0	8140-x-y

Available stoma length (cm):

0.8 /1.0 /1.2 /1.5 /1.7 /2.0 /2.3 /2.5 /2.7 /3.0 /3.5 /4.0 /4.5 /5.0



## MIC-KEY<sup>®</sup> FEEDING TUBE ACCESSORIES

	PRODUCT CODE	
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Medication Port and Right-angle Connector	30 cm	0141-12
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Medication Port and Right-angle Connector	60 cm	0141-24
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Medication Port and Right-angle Connector	5 cm	0142-02
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Straight Connector	30 cm	0143-12
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Straight Connector	60 cm	0143-24
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Single Connection and Right-angle Connector	30 cm	0144-12
MIC-KEY <sup>®</sup> Extension set ENFit <sup>®</sup> with Single Connection and Right-angle Connector	60 cm	0144-24
MIC-KEY <sup>®</sup> Stoma Measuring Device	-	98460



1. Patient Safety Alert NPSA/2011/PSA002. 2. Jacob A. MD, et. al., Safety of the one step PEG Button in children, J. Pediatr., 2015.

The enclosed guidelines are recommendations for appropriate care. These should only be followed when no instructions are available from the healthcare professional or the local hospital. The Avanos guidelines can under no circumstances overrule the local medical or nursing guidelines. The end responsibility for the treatment should remain with the healthcare professional. There are inherent risks in all medical devices, please refer to the product labeling for Indications, Cautions, Warnings and Contraindications.

# AVANOS

For more information, please send an email to [customerservice.uk.ie@avanos.com](mailto:customerservice.uk.ie@avanos.com) or visit [www.avanos.co.uk](http://www.avanos.co.uk).

[www.mic-key.com](http://www.mic-key.com)

<sup>®</sup> Registered Trademark or Trademark of Avanos Medical, Inc., or its affiliates. © 2019 AVANS. All rights reserved.  
ENFit<sup>®</sup> is a Registered Trademark or Trademark of Global Enteral Device Supplier Association, Inc. Used with permission. HC219-00-UK